

# PURELL® VERSAHOLD™ Point of Care Bracket

## Installation Instructions

Please read all instructions before installing and using the PURELL VERSAHOLD Bracket

### Included in package (5704-06-BLU):

- A. (1) Bottle Holder
- B. (1) Clamp
- C. (1) Adapter Tray
- D. (1) Instruction Sheet

### Range of Clamp:

Round vertical surfaces 5/8" to 1 1/2" (15.9 – 38.1mm)

Flat horizontal or vertical surfaces 1/2" to 1 1/2" (12.7 – 38.1mm)



Bottle Holder



Clamp



Adapter Tray

Use with 12 fl oz gel square bottles only – does not fit all 12 fl oz SKUs

### Bracket features referenced in installation:




Clamp Release Button



Thumbscrew Knob



Latch

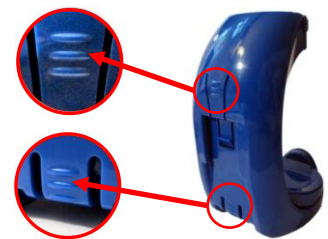

Release Latches

### Step-by-step Instructions:

#### Determine the location and orientation for mounting the bracket

Bracket can be attached with the clamp on the bottom of the bottle holder in 4 different configurations for horizontal mounting or on the backside in 3 different configurations for vertical mounting or attaching to upright surfaces.



Bottle holder can be attached to clamp to face four different directions for use on bedside tables, mobile work stations, shelves, medical carts and more.



Position clamp right or left for IV poles, food carts, vertical shelves, etc.




Position clamp down for upright surfaces, such as bed rails or work dividers. Note: Round horizontal bars require solid support at the bottom of the bracket to prevent bracket from rotating downward.

# PURELL® VERSAHOLD™ Point of Care Bracket Installation Instructions


## Prepare to attach clamp to surface or fixture

1. Prior to attaching clamp, clean surface or fixture area for a better grip.

2. Press and hold release button while pulling the clamp mouth fully open

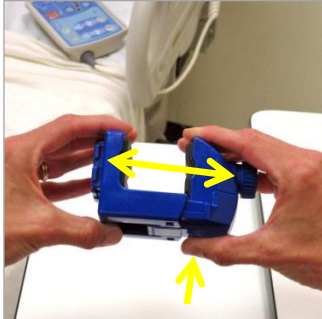

3. Ensure thumbscrew knob is fully open by turning it counter clockwise until it stops rotating

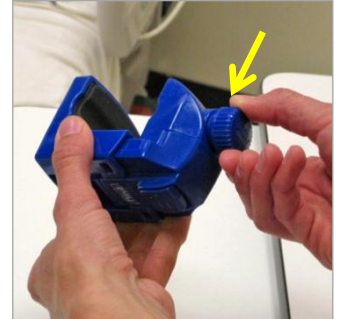


## Attach clamp to surface or fixture

1. Push clamp onto surface ensuring it is fully engaged



2. Squeeze the clamp onto surface with both hands until tight

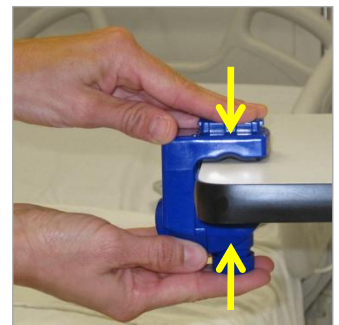


3. Turn thumbscrew knob clockwise, 3-4 complete rotations, to tighten the clamp to its maximum force



*Steps 1-3 for vertical attachment*

– for round vertical fixtures like an IV pole, position clamp on indentations in black inside grip area.

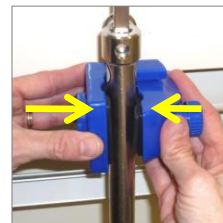

## Attach bottle holder to clamp

1. Determine orientation for bottle holder



2. Slide bottle holder onto clamp until it clicks into locked position



*For vertical mounting, slide down*



## PURELL® VERSAHOLD™ Point of Care Bracket Installation Instructions

### Open latch and insert PURELL bottle

1. To open the latch, push in and down at the same time on the upper left corner



2. For 535 mL bottles, remove adapter tray if present – grasp bottom back section and pull up firmly

For 12 fl oz bottles, insert adapter tray if not present - taller portion goes towards the back - push firmly until it clicks



3. Load bottle, bottom in first and close latch by pushing in



Step 3 for 12 fl oz bottle:



### Remove bottle holder

1. Fully press in release latch and slide bottle holder off clamp


-Press **bottom latch release** when clamp is installed on bottom of holder

-Press **top latch release** when clamp is attached to backside of holder



### Remove clamp

1. Loosen thumbscrew knob by turning it counter clockwise until it stops rotating

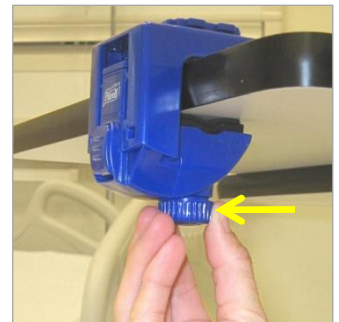

2. Press release button and pull clamp completely open

