

VANGARD™

DISINFECTANT SPRAY

PRODUCT DESCRIPTION:

Vanguard™ Disinfectant Spray dries quickly and leaves no film. An effective disinfectant for such surfaces as: door handles, bed frames, springs, wheelchairs, light switches, linen carts, hampers, telephones, clean up carts, bassinets, dressing carts, toilets, urinals, showers, bathtubs, counter tops, table tops, sinks, waste containers, lockers. With a contact time of 10 minutes at 25°C, this product is effective on hard, non-porous, inanimate surfaces, such as those mentioned above, against the following organisms: Staphylococcus aureus, Trichophyton mentagrophytes, Pseudomonas aeruginosa, Mycobacterium tuberculosis, Salmonella choleraesuis, and HIV-1 (AIDS Virus)*.

Use as disinfectant in: patient rooms, utility areas, dressing rooms, treatment rooms and hospital rooms. This product has been specifically designed for use in hospitals, veterinary clinics, animal shelters, kennels, hospital emergency rooms, ambulance and patient transfer vehicles, prisoner holding areas, offices, funeral homes, restrooms, medical and dental offices, and day care facilities.

NOTE: KILLS HIV-1 (AIDS VIRUS) ON PRE-CLEANED ENVIRONMENTAL SURFACES/ OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood/body fluids can be associated with the potential for transmission of human immunodeficiency virus Type-1 (HIV-1)(associated with AIDS).

*SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV OF SURFACES/OBJECTS SOILED WITH BLOOD AND BODY FLUIDS.

PERSONAL PROTECTION: When handling items soiled with blood or body fluids use appropriate barrier protection such as disposable latex gloves, gowns, masks, or eye coverings.

CLEANING PROCEDURE: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant.
DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to Federal, State and local regulations for infectious waste disposal.
DO NOT USE ON POLISHED WOOD FURNITURE OR RAYON FABRICS.

FEATURES:

Effective against the following pathogenic bacteria: Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella choleraesuis, Mycobacterium tuberculosis.

Fungicidal against: Trichophyton mentagrophytes (the athletes foot fungus when used in locker rooms, showers, exercise areas, dressing rooms, etc.)

Virucidal against: HIV-1 (AIDS Virus)

PRODUCT SPECIFICATIONS:

pH	9.5 ± 0.3
Appearance	Pale Yellow
Fragrance	Fresh
Physical State	Liquid Under Pressure
Freezing Point	About 32°F
Solubility In Water	Dispersible
Warehouse Storage (NFPA)	Level 2 Aerosol
Flash Point	68°F

ACTIVE INGREDIENTS:

o-phenylphenol	0.210%
ethyl alcohol	69.623%
Other Ingredients	30.167%
TOTAL	100.000%

DILUTIONS	HEALTH	FLAMMABILITY	REACTIVITY
CONCENTRATE	NFPA 1	HMIS 3	NFPA 0
CONCENTRAR	HMIS 1	NFPA 3	HMIS 0
PROPORCIÓN	SALUD	INFLAMABILIDAD	REACTIVIDAD

0	1	2	3	4
Minimal	Slight	Moderate	Serious	Severe
Mínimo	Mínimo	Moderar	Grave	Drástico

EPA REGISTERED:

EPA Reg. No. 211-32-1124


EPA Est. No. 62511-KS-1

DATA TECH SHEET

DIRECTIONS

USE INFORMATION


It is a violation of Federal Law to use this product in a manner inconsistent with labeling. Shake well before using.

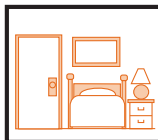

BEDROOMS



KITCHENS

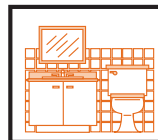

BATHROOMS

DIRECTIONS FOR USE:

AS A MILDEW PREVENTATIVE: Spray surface until moist.

AS A DISINFECTANT: Clean surface to remove all soil. Hold can upright 6 to 8 inches from the surface. Spray 2 to 3 seconds or until surface is wet. Allow to stand 10 minutes before wiping, if wiping is required.

FIRST AID:

WARNING: In case of contact, immediately flush eyes or skin with plenty of water. Get medical attention if irritation persists. Avoid contamination of food.

HANDLING AND STORAGE:

PHYSICAL OR CHEMICAL HAZARDS: Flammable. Contents under pressure. Keep away from heat, sparks, and open flame. Do not puncture or incinerate container. Exposure to temperatures above 130°F may cause bursting.

STORAGE AND DISPOSAL

Storage: Store in a cool, dry area away from heat or open flame.

Pesticide Disposal: Do not contaminate water, food, or feed by storage or disposal. Wastes resulting from the use of this product may be disposed of on site or at an approved waste disposal facility.

Container Disposal: Replace cap and discard in trash. Do not incinerate or puncture. **KEEP OUT OF REACH OF CHILDREN.**

PERSONAL PROTECTION:

Respiratory Protection: Not needed in normal use.

Ventilation: Use with adequate ventilation.

Eye Protection: Safety glasses are recommended.

Protective Gloves: Protective gloves are optional.

Other Protective Clothing or Equipment: Wear clothing as necessary to prevent skin contact.

TRANSPORTATION INFORMATION:

U.S. DOT - Proper Shipping Name: Consumer Commodity, ORM-D

Marine Pollutant: No.

MSDS FAX RETRIEVAL
Read MSDS before using this product

800-551-0101

OBTENCIÓN DE HDSM POR FAX
Leer la HDSM antes de utilizar este producto

FOR CHEMICAL EMERGENCY,
SPILL, LEAK, FIRE, EXPOSURE OR ACCIDENT
CALL CHEMTREC DAY OR NIGHT

800-424-9300

PARA EMERGENCIAS QUÍMICA DERRAMES,
ESCAPES, FUEGO, EXPOSICIÓN O ACCIDENTE
LLAMAR DE DÍA O DE NOCHE A CHEMTREC

TECH HOTLINE
800-810-4829
LINEA DE TÉCNICA