**LUMISCOPE®**  
*For The Quality Of Life™*

**UPPER ARM AUTOMATIC  
DIGITAL BLOOD PRESSURE MONITOR  
MODEL 1133**

**INSTRUCTION MANUAL  
&  
BLOOD PRESSURE RECORD LOG**

**Read and understand these instructions before using the Lumiscope Blood Pressure Monitor. Keep these instructions for future use.**

**Note: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com)**

## CONTENTS

1	INTRODUCTION.....	3
	SAFETY .....	3
	PARTS AND COMPONENTS .....	4
2	SYMBOLS .....	4
3	GENERAL .....	5
	INTENDED USE .....	5
	PRINCIPLE OF OPERATION .....	5
4	TIPS ON TAKING BLOOD PRESSURE MEASUREMENT.....	6
5	BATTERY INSTALLATION.....	7
6	USING THE DEVICE WITH THE INCLUDED AC POWER ADAPTER.....	8
7	SETTING THE DATE AND TIME .....	9
8	CORRECT POSTURE FOR MEASUREMENT.....	10
9	ASSEMBLING AND APPLYING THE CUFF .....	11
10	TAKING A MEASUREMENT .....	12
	AUTOMATIC INFLATION.....	13
	RAPID DEFLATION DURING MEASUREMENT .....	13
11	MEMORY FUNCTION.....	14
	MEMORY RECALL .....	14
	DELETING THE MEMORY .....	15
12	WHO BLOOD PRESSURE CLASSIFICATION.....	16
13	ERROR AND LOW BATTERY INFORMATION.....	17
14	CARE, STORAGE, REPAIR AND RECYCLING .....	18
15	TROUBLESHOOTING .....	19
16	SPECIFICATIONS.....	20
17	TWO-YEAR LIMITED WARRANTY .....	21
18	BLOOD PRESSURE RECORD LOG .....	22
	INSTRUCCIONES EN ESPAÑOL .....	25

Graham-Field® and Lumiscope® are registered trademarks of GF Health Products, Inc.

Packaging, warranties, products, and specifications are subject to change without notice. GF Health Products, Inc. is not responsible for typographical errors.

## 1 INTRODUCTION

### Safety

Please note the following special statements, used throughout this manual, and their significance:

**⚠ WARNING:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.

**▲ CAUTION:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in minor personal injury or product/property damage.

*Note: Provides application recommendations or other useful information to ensure that you get the most from your product.*


**⚠ WARNING:** Important! Read and understand these instructions before using the Lumiscope Blood Pressure Monitor. If you do not understand any part of these instructions, contact your medical professional or GF dealer for direction in the use of this product.

**⚠ WARNING:** If components are damaged or missing, contact your GF dealer immediately. DO NOT use substitute parts.

**⚠ WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

## Parts and Components

1	Main Body
2	Display
3	Air Connector
4	Tube Plug
5	Air Hose
6	Cuff (fits 9" - 14" circumference arm)
7	D-Ring
8	AC Adapter
9	Button "M1"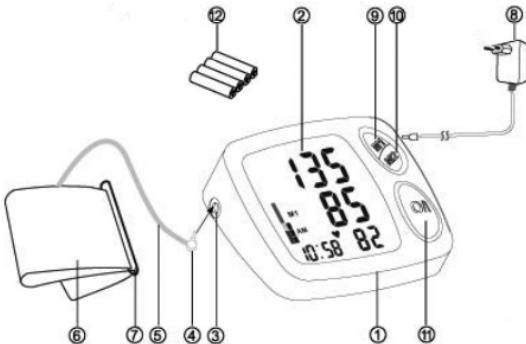

10	Button "M2"
11	Button "O/I"

12	Batteries
13	Carry case (not shown)

## 2 SYMBOLS

Symbol	Meaning
	Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. The device, accessories and packing must be disposed of as waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal.
	CE marking in conformity with EC directive 93/42/EEC
	Attention, consult accompanying documents
	Type B Applied Part

### **3 GENERAL**

#### **Intended Use**

This instruction manual is intended to assist the user in safe and efficient operation of the **Upper Arm Automatic Digital Blood Pressure Monitor Model 1133** (hereinafter: device). The device must be used in accordance with the procedures described in this manual. It is important to read and understand the entire manual, especially the <TIPS ON TAKING BLOOD PRESSURE MEASUREMENT> section.

This device is intended for the noninvasive measurement of systolic and diastolic arterial blood pressure and pulse rate in adults (age 15 and above). Consult your physician before using this device on children or persons with arrhythmia.

#### **Principle of Operation**

This device uses oscillometric technology with Fuzzy Algorithm to measure the arterial blood pressure and pulse rate. The cuff is wrapped around the upper arm and inflated automatically. The sensor of the device catches weak fluctuation of the pressure in the cuff produced by extension and contraction of the artery of the arm in response to each heartbeat. The amplitude of the pressure waves is measured, converted in millimeters of the mercury column, and is displayed by digital value.

*Note: This device cannot provide reasonable accuracy if used or stored at temperature or humidity beyond the range stated in the <SPECIFICATIONS> section of this manual.*

## 4 TIPS ON TAKING BLOOD PRESSURE MEASUREMENT

1. It is necessary to know that arterial blood pressure is subjected to sharp fluctuations. The level of the arterial blood pressure depends on many factors. Generally arterial blood pressure is lower in the summer and higher in the winter. Arterial blood pressure changes with atmospheric pressure and is affected considerably by many factors, e.g. physical loads, emotional excitability, stress, meals, etc. Medicines, drinking, and smoking can affect blood pressure. When blood pressure is measured in the hospital, the value is sometimes higher than when taken at home. This is believed to be associated with anxiety experienced during the hospital/clinic visit. Blood pressure may be elevated when taken in low temperatures. If this device was stored at a low temperature, it is necessary to leave it at room temperature for at least 1 hour, otherwise the measurement can be inaccurate. Blood pressure varies by age and from person to person, and it is recommended to record blood pressure daily; then you can check with your physician to find out what normal blood pressure is for you.

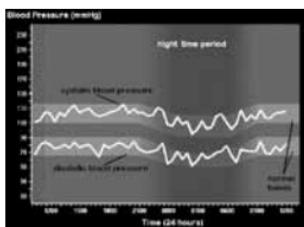
Fig. 4-1

The illustration above is from The British Hypertension Society

2. Patients with cardiovascular diseases: Take measurement under doctor's instruction.

**⚠ WARNING: Under no circumstances should you alter the dosage of any drug prescribed by your doctor!**

3. Accurate measurement of blood pressure may be difficult in people with serious arteriosclerosis, weak pulse, or in patients with irregular heart rates. Please consult a qualified physician to interpret your blood pressure readings.
4. Keep quiet and relax during measurement. To obtain accurate readings, wait one hour after a meal or smoking before taking a measurement.
5. Wait at least 5 minutes before taking the next measurement. Patients with atherosclerosis or diabetes require longer intervals (10-15 minutes), since vessel elasticity may have decreased in these pathologies.
6. This device is supplied with a standard cuff which accommodates arm sizes of 9-14 inch circumference.

*Note: DO NOT use a cuff other than the original cuff contained in this kit.*

## **5 BATTERY INSTALLATION**

1. Open the battery cover and install four alkaline AA batteries in the battery compartment as indicated. Check polarity and ensure the batteries are correctly positioned;
  2. Close the battery compartment cover.
- Replace the batteries when the battery replacement indicator “  ” displays or when the device does not turn on when button “O/I” is pressed;
  - Use only alkaline AA batteries, do not use rechargeable batteries;
  - Use only same-age, same-type batteries together. Replace all batteries simultaneously;
  - If the device is to be unused for long time, take out the batteries and store in a dry environment;
  - Do not leave uncharged batteries in the device.

## 6 USING THE DEVICE WITH THE INCLUDED AC POWER ADAPTER

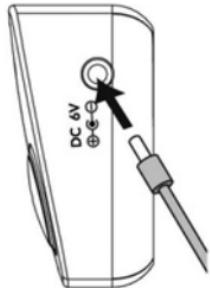
Fig. 6-1

Instead of batteries, the included AC power adapter can be used as the power supply. The AC power connector is located on the right side of the device as shown in Fig. 6-1.

## 7 SETTING THE DATE AND TIME

The Date and Time function creates the ability to document the date and time each measurement is taken. This device has a 12-hour time format. The icon “AM” indicates the time of Ante Meridiem and the icon “PM” indicates the time of Post Meridiem. To display accurate date and time, the user should set the date and time correctly before the first use of this device.

The procedure for setting Date and Time is as follows:

Fig. 7-1

1. When the device has batteries installed, the display will show as in Fig. 7-1;

Fig. 7-2

2. Press and hold button “M1”, then press button “O/I simultaneously”, and the year number will flash;
3. Press button “M1” or “M2” again to increase or decrease the number, and press button “O/I” for confirmation;

Fig. 7-3

4. When the year setup is confirmed, the month number will flash automatically. Follow the same steps as above to set month, day and time;
5. Press button “O/I” to finish setup. If you want to change the date and time, repeat steps 2, 3, and 4.

## 8 CORRECT POSTURE FOR MEASUREMENT

Fig. 8-1

1. Sit down and place your left elbow on a table so that the upper arm cuff, when applied, will be at the same level as your heart.

*Note: The level of your heart is slightly below your armpit;*

2. Extend the left arm, palm up. Loosen and roll up left sleeve so that the blood pressure monitor cuff will be in direct contact with the left arm during measurement.
3. Relax your entire body, especially the area between your elbow and fingers.
4. If the cuff is not at the same level as the heart, or if you cannot keep your arm completely still throughout the measurement, use a soft object such as a folded towel to support your arm. Apply the cuff to the left arm as explained in the instructions on the following page.

## 9 ASSEMBLING AND APPLYING THE CUFF

Fig. 9-1

Fig. 9-2

Fig. 9-3

Fig. 9-4

1. Unwrap the arm cuff, leaving the index end of the cuff through the D-ring of the cuff, as shown in Fig. 9-1 above.
2. Ensure any clothing which restricts the upper left arm has been removed. Wrap the cuff around the upper arm. The tubing should point in the direction of the palm of the hand as shown in Fig. 9-6 below. Position the artery mark over the main artery (on the inside of the arm) of the upper arm. Wrap the cuff around the upper left arm using the end of the cuff so that the lower edge of the cuff is approximately 0.8-1.2 inches from the inner side of the elbow joint as shown in Fig. 9-3 above.
3. The right upper arm may be used instead of the left upper arm; however, it is necessary to know that the readings may differ 5 ~ 10 mmHg between left arm and right arm.
4. Press the cuff to make sure that it is attached securely as shown in Fig. 9-4 above. The cuff should not be too tight or too loose. Two fingers should be easily inserted between cuff and upper arm.
5. If the Index marker points into the Normal range (9"-14" arm circumference) indicated at the edge of the cuff, the cuff is suitable for use. If the Index line falls out of the Normal range, a different size cuff may be required. Call 1-800-672-8293 for more information regarding additional cuffs.

Fig. 9-5

Fig. 9-6

Fig. 9-7

## 10 TAKING A MEASUREMENT

*Note: Before taking a measurement, take 3 ~ 5 deep breaths and relax. In order to obtain accurate measurement results, remain seated, keep your arm and hands still, and do not talk during measurement.*

Fig. 10-1

Fig. 10-2

Fig. 10-3

1. Insert the tube plug into the air connector on the side of the main body of the device.
2. Press button “O/I”; all symbols will display for 2 seconds as shown in Fig. 10-1; you will hear two short beeps; then “0” will display. The automatic pump will then begin to inflate the cuff. Generally the pressure will reach 190 mmHg as shown in Fig. 10-2 before starting to deflate.
3. As the pump then stops inflating and begins to deflate, pressure will decrease gradually, as shown in Fig. 10-3. During deflation the user's blood pressure and pulse will be calculated.

Fig. 10-4

4. When measurement calculation is complete you will hear a long beep, the remaining air in the cuff will deflate quickly, and the blood pressure and pulse measurements will display. Date and time will display alternately as shown in Fig. 10-4; at the same time, “M1” and “M2” will flash to remind the user to record the measurement.

Fig. 10-5

5. Press button “M1” or “M2” to record the measurement in “M1” or “M2” memory, respectively. For example, if button “M2” is pressed, the display will alternate as shown in Fig. 10-5. If neither button “M1” nor “M2” is pressed, the measurement will not be recorded.
6. Press button “O/I” to return to standby mode. Wait for at least 5 minutes before taking another measurement. If the device remains unused for 3 minutes, it will return to standby mode automatically.

### **Automatic Inflation**

This device has 4 levels of inflation pressure: 190 mmHg, 230 mmHg, 270 mmHg and 300 mmHg. When 190 mmHg is insufficient to obtain accurate measurement, or arm movement occurs, the device will automatically inflate to an appropriate pressure level in order to obtain the blood pressure measurement.

### **Rapid Deflation During Measurement**

If you do not feel well during measurement, or want to stop measurement for any reason, press button “O/I” to quickly deflate the cuff and return to standby mode.

## 11 MEMORY FUNCTION

### Memory Recall

- Model 1133 can store 120 sets of readings total (60 sets in “M1” and 60 sets in “M2”) and will automatically calculate the average of the last 3 readings for “M1” or “M2”. When memory is full, the oldest reading will be replaced by the newest. Memory will not clear even if the batteries are removed;

Fig. 11-1

- After a measurement is finished, or when the device is in standby mode, the user can press button “M1” or “M2” to recall memory. Press button “M1” to display the average value of the last 3 measurements as shown in Fig. 10-1;

Fig. 11-2

- Press button “M1” again, the device displays “01”; then after 1-2 seconds the latest measurement will display as shown in Fig. 11-2;
- Press button “M1” again, the device displays “02”; then after 1-2 seconds the second-to-the-last measurement will display.

5. The same procedure can be used to recall “M2” measurements.

### **Deleting the Memory**

Fig. 11-3

To clear memory: After a measurement is finished or when the device is in standby mode, press and hold button “M1” or “M2” for at least 5 seconds. The device will display “Clr”, which means the stored readings for “M1” or “M2” have been deleted, as shown in Fig. 11-3.

## 12 WHO BLOOD PRESSURE CLASSIFICATION

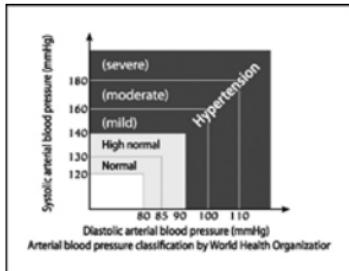
Fig. 12-1

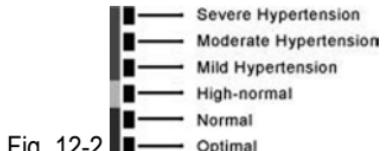
Fig. 12-2

Fig. 12-3

This device features a 3-color indicator to display the blood pressure level in accordance with the WHO (World Health Organization) classification. The World Health Organization (WHO) has established standards for assessment of blood pressure, regardless of age, as shown in Fig. 12-1: The indicator displays a black segment next to the green, yellow and red permanently-displayed segments to indicate the WHO classification after a blood pressure measurement is taken. The green segment corresponds with **Optimal** or **Normal** blood pressure classification, the yellow with **High Normal**, and the red with **Mild**, **Moderate**, or **Severe Hypertension**, as shown in Fig. 12-2.

For example, if your blood pressure is 135 mmHg (Systolic Pressure) over 78 mmHg (Diastolic Pressure), according to the World Health Organization standard, your blood pressure classification is **High Normal**. Three black bars will display next to the colored segment as shown in Fig. 12-3.

*Note: If the systolic blood pressure and diastolic blood pressure fall into different categories, use the higher value for classification.*

## 13 ERROR AND LOW BATTERY INFORMATION

INDICATION	POSSIBLE REASON	CORRECTION METHODS
	<p>The cuff is positioned incorrectly</p> <p>The tube plug is not securely attached to the main body</p> <p>Movement of arm (or hand) or talking during measurement</p> <p>The cuff is not inflated to sufficient pressure</p> <p>Arrhythmia</p>	<p>Ensure cuff is correctly positioned; repeat measurement</p> <p>Ensure tube plug is securely attached to main body</p> <p>Remain still and quiet; repeat measurement, following recommendations of this manual</p> <p>Repeat measurement; inflate cuff to at least 40 mmHg higher than the expected systolic pressure</p> <p>Consult your physician</p>
	The batteries are weak	Replace all four batteries with new ones

## **14 CARE, STORAGE, REPAIR AND RECYCLING**

1. It is necessary to protect this device against high moisture, direct sunlight, shock, alcohol and gasoline.
  2. Remove the batteries if the device is to be stored for a long period of time. Keep the batteries away from children.
  3. Prevent sharp objects from coming into contact with the cuff. Do not stretch or twist the cuff.
  4. Use only clean, soft, dry cloths to clean the device.
  5. The cuff is sensitive and must be handled with care. Clean the cuff cover with a clean, damp cloth. Ensure the cuff is dry before taking a new measurement.
- ▲ CAUTION: Do not wash the cuff's inner bladder.**
6. Since neither this device nor batteries are household waste, follow your local recycling rules and dispose of them at appropriate collection sites.

## 15 TROUBLESHOOTING

SYMPTOM	CHECK POINT	REMEDY
No display when button "O/I" is pressed	The batteries need to be replaced Battery polarities are incorrectly positioned The battery contacts are corroded	Replace the batteries with new ones Install the batteries in correct position Clean the battery terminals with a dry cloth
Inflation stops then starts again	See <TAKING A MEASUREMENT/Automatic Inflation> Did you talk or move your arm (or hand) during measurement?	See <TAKING A MEASUREMENT/Automatic Inflation> Keep quiet and still during the measurement
The reading is extremely low or high	Ensure the cuff is positioned at the same level as the heart Is the cuff wrapped correctly? Did you strain your arm (or hand) during measurement? Did you talk or move your arm (or hand) during measurement?	Ensure your posture is correct Ensure the cuff is wrapped correctly Relax during measurement Keep quiet and still during the measurement
Pulse rate is too low or too high	Did you talk or move your arm (or hand) during measurement? Did you take your measurement right after exercise?	Keep quiet and still during the measurement Take measurement again after resting for at least 5 minutes
The batteries wear out too quickly	Faulty batteries used	Suggestion: Use alkaline batteries of known manufacturers

## 16 SPECIFICATIONS

Model	1133
Size (L x W x H)	5" x 3.94" x 1.97" (127 mm x 100 mm x 50 mm)
Weight (without batteries)	~ .55 lb (252 g)
Measuring method	Oscillometry
Measuring range	40 to 260 mmHg (blood pressure) 40 to 160 beats/minute (pulse rate)
Measuring accuracy	± 3 mmHg for systolic and diastolic pressure ± 5% of the reading for the pulse rate
Inflation	Automatic
Rapid deflation	Automatic electronic valve
Batteries	4 alkaline AA x 1.5V
AC Adapter	6V, 600mA
Memory	2 x 60 sets of measurements
Temperature and humidity	Operation: 50°F to 104°F (10°C to 40°C), RH ≤ 85% Storage: -4°F to 122°F (-20°C to 50°C), RH ≤ 85%
Cuff size	Fits arm circumference size 9" - 14" (229 mm - 356 mm )
Complete kit	Main body, cuff, 4 x AA batteries, AC adapter, instruction manual, carry case

## **17 TWO-YEAR LIMITED WARRANTY**

GF Health Products, Inc. guarantees this product free from defects in material and workmanship for a period of two years from the date of purchase, except as noted below:

This product warranty does not cover damage caused by misuse or abuse, the attachment of any unauthorized accessory, alteration to the product, or any other conditions whatsoever that are beyond the control of GF Health Products, Inc. GF Health Products, Inc. shall not be responsible for any type of incidental, consequential, or special damage. All implied warranties, including, but not limited to those implied warranties of fitness and merchantability, are limited to the total duration of two years from the original date of purchase.

To obtain warranty service on your Blood Pressure Monitor, please contact the Repair Department at 1-800-672-8293. There is a \$15 fee for return shipping and handling. Please make checks payable to GF Health Products, Inc. Upon receipt, GF Health Products, Inc. will repair or replace, as appropriate, this blood pressure monitor and return it to you.

Warranty is solely through the Repair Department. Service of this product by anyone other than the Repair Department voids warranty.

## 18 BLOOD PRESSURE RECORD LOG

Date	Systolic (mmHg)	Diastolic (mmHg)	Pulse (beats/minute)

Date	Systolic (mmHg)	Diastolic (mmHg)	Pulse (beats/minute)

Date	Systolic (mmHg)	Diastolic (mmHg)	Pulse (beats/minute)

Date	Systolic (mmHg)	Diastolic (mmHg)	Pulse (beats/minute)

**GF Health Products, Inc.  
2935 Northeast Parkway  
Atlanta, Georgia 30360**

**telephone: 800-347-5678, 770-368-4700  
fax: 800-726-0601, 678-291-3232**

**GRAHAM-FIELD.  
[www.grahamfield.com](http://www.grahamfield.com)**

**© 2010 GF Health Products, Inc.  
Made in China**

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